

### Pre-fabricated Elbow Extension Brace

*How to measure for a Pre-fabricated Elbow Brace:*

1. Facing the patient, position the elbow in maximum extension and the wrist in neutral pronation/supination. See diagram to check proper positioning for measurement.
2. A circumference measurement should be taken around the arm at each level.

**Level A** - is located 7 inches (17.8 cm) above joint center.

**Level B** - is located at the joint center.

**Level C** - is located 7 inches (17.8 cm) below joint center.

Measurement for Pre-fabricated Elbow Extension Brace					
	Inches			Centimeters	
		circumference		circumference	
Small	A	8 - 11.75	A	20.3 - 29.8	
	B	8 - 11.75	B	20.3 - 29.8	
	C	5.75 - 8	C	14.6 - 20.3	
Medium	A	10.25 - 14.25	A	26 - 36.2	
	B	10.25 - 14.25	B	26 - 36.2	
	C	6.75 - 10.5	C	17.1 - 26.7	
Large	A	12.75 - 16.5	A	32.4 - 41.9	
	B	12.75 - 16.5	B	32.4 - 41.9	
	C	8 - 11.75	C	20.3 - 29.8	

